

USB IrDA adapter installation

For installing the driver of the IrDA adapter please perform the following steps (the same description can be found in the Installation Users Manual).

Important! Please note that the IrDA adapter must not be plugged in the PC before or during installation! To complete the driver installation the computer must be restarted!

1. Please choose your language (in this case English)
2. Click “Next” to run the procedure.
3. Click “Install” to exit the install program.
4. Restart the computer to complete the install procedure. (Without restarting the PC the driver will not work properly. All open data should be saved before restart!)
5. After restart plug in the IrDA adapter. (Do not plug in the adapter before or during the installation!)
6. The IrDA device is ready to communicate.